



## Blue Top Tubes (3.2% Sodium Citrate)

Blue top tubes (3.2% Na Citrate) should be filled as below. Tubes should be mixed by gentle inversion immediately after drawing.

### Manufactured by BD

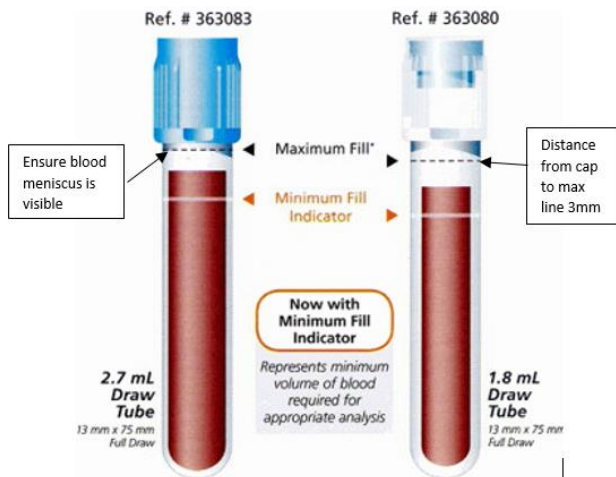

### Manufactured by Greiner

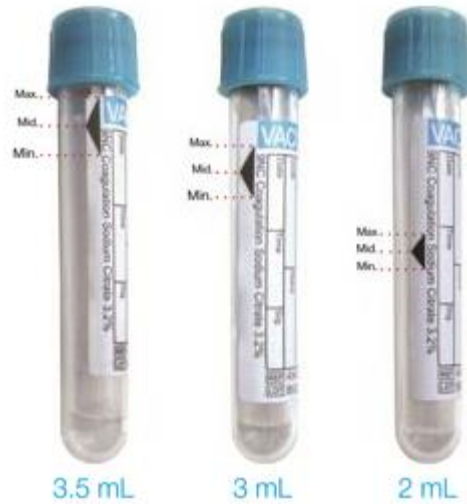

Sodium citrate microtainers should only be used for venous or arterial blood. The tube should be filled with 0.9 ml blood drawn with a syringe. The tube should be filled to the 1.0 ml line. Tubes should be mixed after filling. Ensure that the blood actually moves from end to end in the tube.



Effective 4/5/2022